

## Fitting Guide


# **SCOOTBOOT** TheraRide



**Thank you for choosing Scootboot TheraRide**

It is important to ensure the fit is correct before using your Scootboot TheraRide.

# Scootboot TheraRide Parts

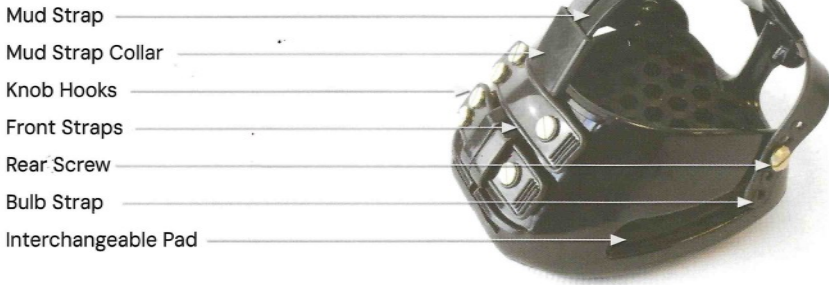

## You will need

- Clean flat surface (eg towel, cardboard, rubber mat)
- Phillips head screwdriver (minimum 20cm/8" length)
- Flat head screwdriver
- At least half an hour for your initial setup.



## Fitting The TheraRide

- Undo the Front Straps and release the Mud Straps.
- Pull one Mud Strap side out of the Collar.
- Slip the Scootboot TheraRide onto the hoof and check it is a snug fit.
- Fasten Front Straps and leave Mud Straps open.



## Checking Bulb Strap Tension

- The Bulb Strap must sit firmly over the heel bulbs. If not, remove the Scootboot TheraRide and change the rear screw positions using both screw drivers to tighten.



## Fitting The Mud Strap

- Bring the Mud Straps around the pastern and thread both through the Collar.
- Slide the Mud Strap loops through the bottom Front Strap and secure the Front Strap on the Knob Hook.



- Secure the top Front Strap making sure it sits over the Mud Straps.
- Ensure the Collar is positioned above the Top Front Strap.
- Mud Straps should have a snug fit around the pastern.
- There should be slight upward tension on the lower Front Strap.



### Underrun and High Heeled Hooves

- If the Bulb Strap is sitting too low, try the medium Bulb Strap (included in the accessories bag), or lift it higher by inserting the Bulb Strap Extender.



Bulb Strap Extender

### Clearance Zone

- Ensure there is space between the base of the pastern and the Bulb Strap to prevent rubbing.



Clearance Zone

### Insufficient Clearance Space

- The Bulb Strap can be removed and replaced with the Extender to support the position of the Mud Strap, but only for gentle riding.



### Final Check



1 You can cut excess length from the Bulb Straps once you have confirmed the fit is correct.



2 After riding and confirming the Bulb Strap tension, secure the screws with Loctite222 or nail polish.




3 If not, you must check screws frequently to ensure they are still tight.

## Using other pads with TheraRide

If you are using your own pad ensure it does not exceed the height of the cavity (10mm-3/8") when compressed as this will affect the fit.

The fit and security will not be affected by the TheraRide Pad.

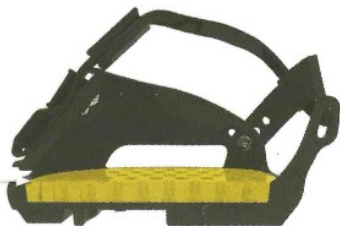

10mm - 3/8"



## Custom Pads

Speak with your hoof care professional who may be able to create or recommend a custom pad using different materials or design to better suit your horse's particular pathology.

Bulb Strap Gaiter



## Bulb Strap Gaiter

The TheraRide comes with a protective neoprene Bulb Strap Gaiter. It is recommended that the Bulb Strap Gaiter be used if your horse has sensitive skin.

MultiPurpose Gaiter



## Do you need added protection and security?

The MultiPurpose Gaiter can be used on the TheraRide for long distance riding or rehabilitation instead of the Bulb Strap Gaiter. If this is used, the Mud Strap must go up a size or place the Mud Strap over the top Front Strap only. (Can be purchased separately at our store.)

## Not sure about the fit of your Scootboot TheraRide?

Please take photos of the fit from all angles before you ride and send them to us. Our fitting experts will evaluate the fit and provide you with further advice.